



# secure sleeper™ vented positioner

## Positions baby as advised for prevention of SIDS

A back-only position is advised for healthy, full-term infants by the Surgeon General, the American Academy of Pediatrics and the Sudden Infant Death Syndrome (SIDS) Alliance to reduce the incidence of SIDS.

## USE INSTRUCTIONS

Place baby on his/her back onto the soft fabric surface, between the parallel side support wedges (bolsters). To properly position the bolsters, snugly secure them against the sides of the baby's torso beneath the armpits. They are designed to fit any baby and should be used to provide added support and comfort as well as keep your baby positioned on his/her back.

## IMPORTANT SAFETY NOTE

Discontinue use when baby is able to roll onto his or her tummy without assistance.

## CARE INSTRUCTIONS

Machine wash separately in cold water, delicate cycle, no bleach. Drip dry. For best results, wash in a laundry bag.



DEX Products, Inc. | [www.dexbaby.com](http://www.dexbaby.com) | 800.546.1996 | 840A Eubanks Drive, Vacaville, CA 95688

© 2009 DEX Products, Inc., secure sleeper™, dexbaby™ and the footprint logo are trademarks of DEX Products, Inc. The vented positioner is protected by U.S. patent numbers 5,193,238, 5,216,772, 5,272,780, and 5,357,642, which are strictly monitored and enforced. Patent pirates will be prosecuted to the full extent of the law. 1209